




Straight Universal Handle 00-8790-010-00

Assembly/Disassembly
Instructions

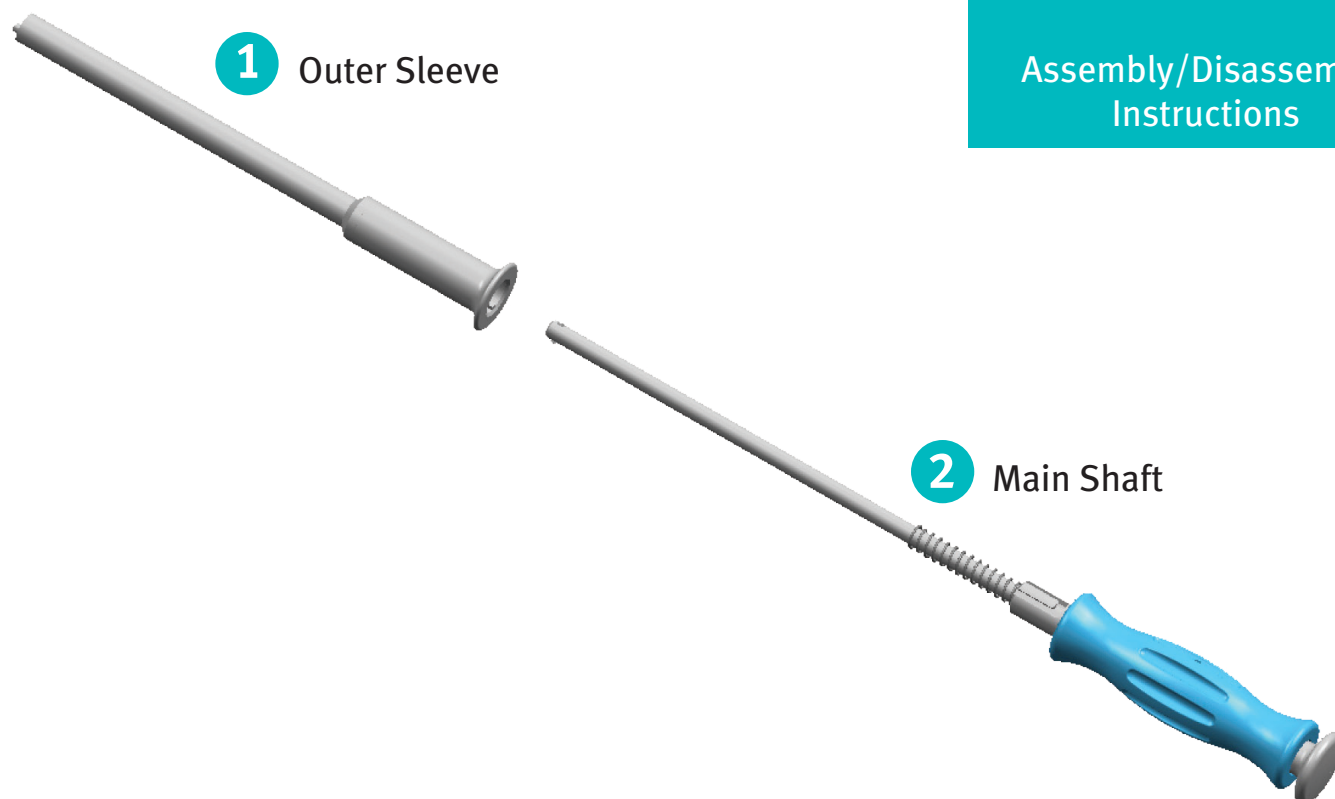

Disassembly Instructions for Straight Universal Handle

1. Pull outer sleeve towards blue handle
2. Rotate outer sleeve until it stops
3. Slide outer sleeve away from blue handle
4. End of disassembly

Note: The outer sleeve must be disassembled from the main shaft for cleaning and sterilization

Assembly Instructions for Straight Universal Handle

1. Align pin in tip of main shaft **2** with slot in outer sleeve **1**.
2. Slide sleeve towards blue handle until it stops
3. Rotate outer sleeve until it stops
4. Release outer sleeve
5. End of assembly



+H124970000301001/\$100610F10H

97-0000-301-00 1003-H22 Printed in USA ©2010 Zimmer, Inc.

