




02.00024.335 NCB® Compression Drill Guide for 5.0mm Screws

Assembly/Disassembly
Instructions

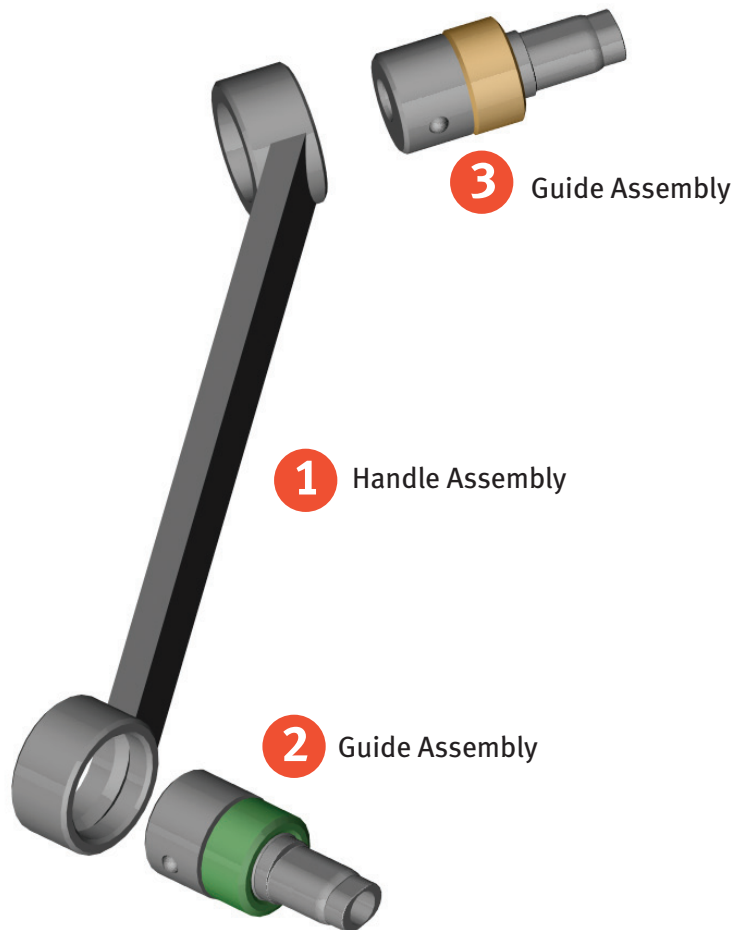

Disassembly Instructions for Drill Guide 5.0mm Screw

1. Pull the Guide Assembly **2** firmly to disengage it from the Handle Assembly **1**.
2. Pull the Guide Assembly **3** firmly to disengage it from the Handle Assembly **1**.
3. End of disassembly.

Assembly Instructions for Drill Guide 5.0mm Screw

1. Push the Guide Assembly **2** firmly inside the Handle Assembly **1** until the spherical ball of the Guide Assembly **2** engages and is free to rotate in the groove of the Handle Assembly **1**.
2. Push the Guide Assembly **3** firmly inside the Handle Assembly **1** until the spherical ball of the Guide Assembly **3** engages and is free to rotate in the groove of the Handle Assembly **1**.
3. End of assembly.



+H124970000526001/\$110224B11P

97-0000-526-00 1102-T04 Printed in USA ©2011 Zimmer, Inc.